

LIFEPAK® CR-T AED Training System Information

What's changing with the CR-T?

Changes mimic the changes made to the actual LIFEPAK CR® Plus AEDs. Visit the *CR Plus* AED upgrade page at www.aedhelp.com/aha/troubleshoot.htm for more information.

Information for U.S. customers

How can I upgrade my existing CR-T?

We are offering a CD with training system upgrade software. Simply insert the software CD into a computer (PC). Instructions will step you through establishing a communication link between the computer and the *CR-T* via a special 9-pin serial port cable and performing the upgrade. Once the training system is upgraded, it will contain all new options and default settings that are consistent with AHA/ERC 2005 Guidelines. You will still be able to change settings as always; refer to the new Operating Instructions on the software CD.

In order to perform this upgrade you will need a computer (PC) with Windows® 98/2000/XP and a 9-pin serial port (a standard connector found on the back of most computers).

Our field service team does not service trainers.

How much will it cost to upgrade my existing CR-T?

The upgrade kit (includes cable and CD) will cost \$30 in the U.S. The kit can be used to upgrade multiple CR-T training systems.

Pricing for CR-T Upgrade Package

Update CD and (1) RS-232 serial Communications cable (Cat. # 43340-000072) \$30.00

When will upgrades be available?

To order your LIFEPAK CR-T upgrade kit, please call 1-800-442-1142.

When will NEW Guidelines-ready CR-T training systems be available?

The new Guidelines-ready CR-T training systems are now available for order and are priced at \$299 each. To order a CR-T training system, please call 1-800-442-1142 and reference catalog number 11250-000073.

Information for customers outside the U.S.

The same upgrade methods available inside the U.S. will also be available in many countries outside the U.S. Localized versions of the new LIFEPAK CR-T AED training systems are available. Upgrade kits will also be available in several languages. See your local sales consultant for details on pricing and availability.

November 2007. Physio-Control.