

## Suggestions for Performance Evaluation

with the LIFEPAK® 20 Defibrillator/Monitor

# Maintenance and Reports

Name: \_\_\_\_\_ Unit: \_\_\_\_\_

Reviewer: \_\_\_\_\_ Date: \_\_\_\_\_

This Performance Evaluation is a suggested assessment of one's ability to maintain and obtain reports from the LIFEPAK 20 defibrillator/monitor series. This evaluation does not cover all information and skills required to operate the device safely and effectively. This evaluation is designed to be completed after viewing the appropriate inservice video or observing an equipment demonstration given by a qualified instructor. For complete information, review the Operating Instructions. References to buttons are indicated in **bold** and screen display messages are indicated in *italics*.

PERFORMANCE EVALUATION	COMPLETE	INCOMPLETE	COMMENTS
1. Daily auto testing a. Ensure defibrillator is plugged in b. Ensure QUIK-COMBO® therapy cable is connected to the QUIK-COMBO test plug or ensure the paddles are securely placed in the paddle wells			
2. User test a. Ensures QUIK-COMBO therapy cable is connected to the QUIK-COMBO test plug or paddles are securely in well b. Presses <b>ON</b> c. Presses <b>OPTIONS</b> d. Selects <i>USER TEST</i> e. Selects <i>YES</i> to confirm start of the user test			
3. Therapy cable integrity test a. Confirms therapy cable is connected to the defibrillator b. For QUIK-COMBO therapy cable: <ul style="list-style-type: none"> <li>• Confirms test plug is connected to therapy cable</li> <li>• Presses <b>ANALYZE</b>, look for the <i>REMOVE TEST PLUG</i> message</li> </ul> c. For hard paddles: <ul style="list-style-type: none"> <li>• Confirms paddles are properly seated in wells</li> <li>• Select 50J and presses <b>CHARGE</b></li> <li>• When fully charged presses <b>SHOCK</b> on paddles, and look for <i>ENERGY DELIVERED</i> message</li> </ul> Note: Inspect defibrillator and accessories per LIFEPAK 20 defibrillator/monitor Operator's Checklist located in Appendix D of the operating instructions			
4. Changing recording paper a. Presses the black door release at the lower right corner of the printer door b. Pulls the slotted edge of the door to open c. Removes the empty paper roll d. Inserts new paper roll onto raised spool brackets with grid facing forward e. Pulls out a short length of paper f. Pushes the rear printer door in and push down on the front printer door to close			
5. Entering data a. Presses <b>OPTIONS</b> and select <i>PATIENT</i> b. Selects the letters and symbols identifying the patients name, ID, etc. c. Presses <b>EVENTS</b> and select events on the menu with the speed dial d. Presses <b>EVENTS</b> and fill in the event information by hand later			

