


Placement of Precordial Leads

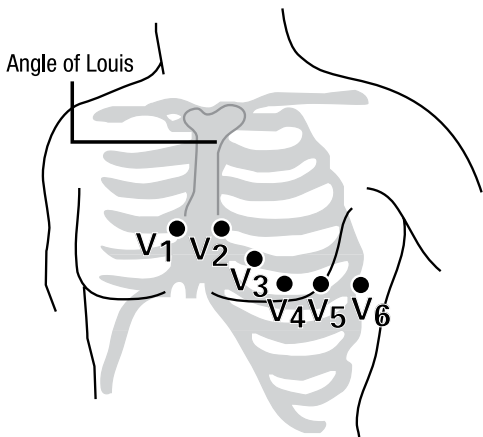

- V1** Fourth intercostal space to the right of the sternum
- V2** Fourth intercostal space to the left of the sternum
- V3** Directly between leads V2 and V4
- V4** Fifth intercostal space at left midclavicular line
- V5** Level with lead V4 at left anterior axillary line
- V6** Level with lead V5 at left midaxillary line

Quick Tips for a Clear Tracing

- Use high quality, fresh electrodes
- Shave hair at electrode site
- For oily skin, clean skin with alcohol pad
- Dry site with brisk rub

Transmitting a 12-Lead ECG

- Establish wireless or direct cable connection
- Press TRANSMIT
- Select DATA
- Select SEND to transmit if REPORT and SITE settings are correct

This information is based on factory default settings and is an abridged reference for precordial lead placement and transmission. Refer to the LIFEPAK® 12 monitor/defibrillator operating instructions for complete directions for use, indications, contraindications, warnings, precautions, and potential adverse events.